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Manufacturer takes responsibility for proper usage of the product only when the product was purchased from qualified REH4MAT employee or at specialized medical store!

REH4MAT reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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Product marked with the CE sign is Class I medical device, of rule 1, consistent with the requirements of Regulation (EU) 2017/745 of the European Parliament and of the Council of 5 April 2017 on medical devices.

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REH4MAT Company introduced quality management system ISO 13485 regarding production, sales and service. The system is certified by Det Norske Veritas.

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INTRODUCTION

PCO (Postural Compression Orthoses) are the medical devices class I, which provide dynamic compression and muscles control with pediatric neurologic disorders such as CP and spina bifida. Dynamic PCO orthoses are made of Lycra and neoprene elements which reduce pressure in bone prominences. Lycra provides deep pressure control abnormal muscle tone. Our PCO orthoses are available in various options: single arm or leg sleeves, suits and shirts. High sensory dynamics and core stability treat the abnormal tone in case of neurologic dysfunctions. Deep compression of particular body parts send more and better sensory information to the brain, increasing proprioception, which in turn decreases high muscles tone and spasticity. PCO orthoses do not limit the range of motion and deep compression of joints normalizes the muscle tonus and reduces involuntary movements, which are often in CP children. Using of Postural Compression Orthoses is recommended for many neuromuscular conditions, mainly CP, with following symptoms: high tone and spasticity, low tone, athetoid, dystonic, dyskinetic, ataxic and mixed presentation. Dynamic PCO orthoses are the best addition to traditional physical therapy and other occupational activities.

INDICATIONS FOR USE

A soft full-body orthosis intended to be used to facilitate correction of muscle dysfunction and postural disturbances associated with central nervous system disease (e.g., cerebral palsy, stroke, traumatic paraplegia) principally by normalization of proprioceptive impulses and inhibition of tone in spastic muscles. It consists of several parts of a suit: vest, belt, shorts, knee pads, shoes and cap, all interconnected with a system of adjustable elastic bands and worn by the patient while participating in a variety of activities (e.g., exercises, gait training, playing) under the supervision of a healthcare provider. It is available in a range of sizes. This is a reusable device. It is believed that this type of Postural Compression Orthoses PCO® and deep compression provides better sensory information to the brain, increases proprioception and child's knowledge about its own body. Due to this fact, dynamic PCO® orthoses are the best solution for improving treatment in case of abnormal tone and decreasing spasticity.

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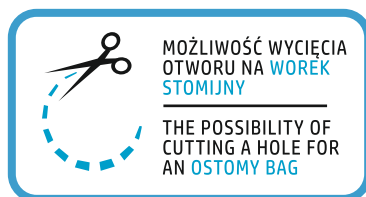
CONTRAINDICATION

The **PCO braces** shouldn't be used in case of:

- Heat induced epilepsy.
- Cardiovascular impairment.
- Neoprene or lycra allergy or tendency to skin inflammation.

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HOW TO WEAR PCO PROPERLY?

High compression of orthosis is required for therapeutic satisfaction. For checking if compression is effective, insert your hand between the Patient's body and orthosis. When you make the slight distance between skin and the brace, you will feel fabric's resistance. If the distance is more than 3-5 cm (1,18-2 inch) it means the PCO orthosis is too loose and we recommend to change size for smaller. If the distance is less than 3-5 cm (1,18-2 inch) it means the brace is too tight and you should try bigger size.

PCO braces usability equals maximum 1 year. After this time, the fabric is overstretched due to many washing. In this case we highly recommend to change the brace for a new one

PCO removing

Always check the skin's condition after removing the orthosis. Redness on your user's skin may be visible and is expected on weight bearing points. If however this redness persists longer than 15 minutes after removing the orthosis, it means the brace is too tight and you should change size for a bigger one.

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HOW THE ORTHOSIS MADE OF MEMORYCAST™ WORKS

Selected PCO products (**PCO-T-03**, **PCO-T-04**) are equipped with thermoplastic panel **MEMORYcast™**, for heat molding, which provides anatomic stabilization. The panel has the same properties as rigid nylon and softens when temperature is approx. 60° C. it's very easy to do that, there is enough to place the brace is hot water. Our thermoplastic panel is safe and biodegradable. Contrary to ordinary plaster cast, high plasticity of **MEMORYcast™** allows to heat and mold it many times according to doctor's recommendations. Furthermore, the brace provides perfect lightweight immobilization. The **MEMORYcast™** panel can be reheated and reshaped at any time, depending on the therapeutic needs.

To shape the **MEMORYcast™** panel, you must:

1. Prepare a dish (big enough to put the product in) and boil water in a kettle.
2. Pour **boiling water** into the prepared dish.
3. Put the back of the vest with the **MEMORYcast™** panel in **boiling water**. Remember to immerse the hole **MEMORYcast™** panel. Wait approx. 1 min.
4. When a product is elastic, take it out of water and drain carefully. **Be careful not to burn yourself with hot water! Check if a brace is not too hot. If it is still hot, wait till its temperature will be safe for body but still plastic.**

Attention! It is important to use boiling water. If water is not hot enough, the plate may be damaged during modeling.

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ATTENTION! There is possible to put the brace on, when the MEMORYcast™ thermoplastic fabric is able to shape but is not too hot.

There is necessary to wait only a few minutes after putting the brace on for cooling the MEMORYcast™. It maintains the patient's anatomic shape.

There is **ABSOLUTELY** necessary to check its temperature before putting in on the skin! Do not put too hot orthosis on the skin, because the risk of burning occurs! In case of the brace is too hot after heating, there is necessary to wait for cooling it down.

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ATTENTION! Selected PCO products (PCO-T-03, PCO-T-04) are equipped with thermoplastic panel MEMORYcast™, for heat molding, which provides anatomic stabilization. There is possible to put the brace on, when the MEMORYcast™ thermoplastic fabric is able to shape but is not too hot. Despite the fact, that MEMORYcast™ panel does not have directly contact with skin, there is **ABSOLUTELY** necessary to check its temperature before putting in on the skin! Do not put too hot orthosis on the skin, because the risk of burning occurs! In case of the brace is too hot after heating, there is necessary to wait for cooling it down.

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Do not:

- Use the product against its purpose of use
- Use the product in the wrong size
- Use the product by people and children with mental disorders without guardian control
- Use the product in case of damage or shape's deformation
- Attempt to service this pillow yourself
- Attempt to change the construction of the product
- Use the accessories which are not recommended by manufacturer

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ATTENTION! For single patient use only.

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ATTENTION! Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.

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ATTENTION! Before using the product, read this manual carefully. Before using the product, check if it has no signs of wear – no damage or tearing of straps, stitching. Only an undamaged product and proper use guarantee its right and safe working. The following information about the product should be kept in order to be able to use the instructions later.

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ATTENTION! Using of the product not according to the manual may cause serious complications

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ATTENTION! Never leave the product in a hot vehicle or other places where the product would be exposed to high temperatures.

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ATTENTION! Product works correctly only when the device is adjusted according to patient's height as well as weight. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

If the symptoms occur (for example skin allergies, intertrigo, bedsores), stop using the product immediately, take the user out of the product and lay them down. If the undesirable symptoms persist, contact a physician immediately. If the symptoms have disappeared, inform the attending physician about this fact in order to make a decision about the further rehabilitation process.

ATTENTION! Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

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PCO BRACES MADE OF LYCRA:

- We recommend to wash braces after each usage in the washing machine at a temperature of under 30°C. We recommend to use the washing machine due to the fact that this process rearranges winding of the fibers and improves the compression. Due to this fact hand washing is insufficient.
- Do not use detergents and bleaches. Do not tumble dry. Do not use fabric softeners.
- Spin gently on low speed.
- Do not iron.
- Do not bleach.
- Do not wring.
- Keep the product away from heat sources. Do not expose them to prolonged sunlight.

PCO BRACES MADE OF NEOPREN:

- Hand wash at below 40°C.
- Do not bleach. Do not tumble dry. Do not use fabric softener.
- Do not spin.
- Do not iron.
- Do not wring.
- Dry flat.
- Keep the product away from heat sources. Do not expose them to prolonged sunlight.

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ATTENTION! The manufacturer is not liable for damages resulting from the use of inappropriate cleaning products. The PCO products should be kept clean and used in accordance with the manufacturer's instructions and this user manual.



ATTENTION! The product should be stored in a clean and dry place. The product should be cleaned systematically as to maintain it in good condition.

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LABELLING:

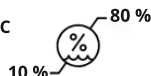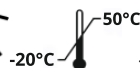	Declaration that the product meets the essential requirements
	Catalog number
	Batch number
	Medical device
	Please read the manual instructions before use
	Attention!
	Producer
	Production date
	Store in a dry place
	Protect from sunlight
	Storage temperature limitation
	Relative humidity storage

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