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The manufacturer holds the ISO 13485 International
Quality Management System certificate.

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ENGLISH

PRODUCT DESCRIPTION

The brace is an excellent solution for effective therapy in case of injury or other conditions. **Reh4Mat** products are appreciated and recommended by specialists (doctors, physiotherapists) and patients. Our products are produced in Poland and made of high quality materials. In case of any questions, feel free to contact us: export@reh4mat.com.

INDICATION

Lower limb orthosis offers stability in case of injury, deformities or neurological conditions. The orthosis provides lower extremity stabilization and improves upright position of user. Lower limb orthoses are widely used in neurology, orthopedics and rheumatology as a support in gait pattern.

CARE AND MAINTENANCE

- Product should be maintain in cleanness and use accosing the manufacturer recommendations. Clean the product systematically
- If the product holds lateral splints or shells and it is possible to dismantle them without any tools, remove them before washing.
- Wash by hands in warm water with soap in temperature of 30°C.
- Sluice carefully, use when completely dry.
- Do not iron, do not chlorinate, do not dry clean, do not tumble dry.
- Keep it in a dry and clean zone.
- Protect from moisture, extreme temperatures, sunlight, dry away from sources of warmth.

SAFETY RULES

- Attention!** The product should be used according the manual's recommendations. If you have any questions, please consult your care professional.
- Attention!** Before using the product, read this manual carefully. Following the instructions included in the manual will provide product's safety and effectiveness. If you have any questions, please consult your care professional.
- Attention!** Using of the product not according to the manual may cause serious complications
- Attention!** Please, keep the manual for future.
- Attention!** Check the product's condition. If the product is damaged, discontinue use and contact the manufacturer. Only an undamaged product and proper use guarantee its right and safe working.
- Attention!** Manufacturer is responsible only for hidden defects in the product or mechanical damages occurred in the natural way of using.
- Attention!** Manufacturer is responsible for product's effectiveness and safety only in case of purchasing the product in the authorized seller.
- Attention!** Pain is always the sign of warning. If the pain maintains or increases, it is advisable to consult your doctor or physical therapist.
- Attention!** For single patient use only.
- Attention!** The product should be used according to the recommendations of a physician or physiotherapist.
- Before first using, check the size of the product. Only good size guarantees safety and effectiveness. A wrong size of the product decreases effect of treatment, increases discomfort and wrong position of the user. Proper use of the brace diagnosis by physician orr physician therapist.
- Control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occuring, it is necessary to contact a physician.
- Chronic using the orthosis may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a physiotherapist as physician has recommended.
- In case of varicose veins, Deep Vein Thrombosis (DVT), oedema the using of orthosis is allowed only after contact with physician.
- Use the orthosis during night should by done only if this was prescribed by your doctor.
- Before using remove package and accessories like papers, cards or labels.
- Do not use the product against its purpose of use.
- Do not use the product in the wrong size.
- Do not use the product by people and children with mental disorders without guardian control.
- Do not use the product in case of damage or shape's deformation.
- Do not attempt to service this product yourself.
- Do not use the accessories which are not recommended by manufacturer.