AKVO

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REH4MAT reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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1. On this water-resistant base AKVO™ you can install the AKVOSEGO™ or AKVOLITO™ bath chairs. Bath chair installed on the base is the great aid which you can use during the shower or in bath tub.
2. Before using the base for the first time, the user (or caregiver) is obliged to familiarize himself/herself with the user manual. The product has to be used with compliance to guidelines included in this user manual! Before every using, there is necessary to prepare the patient and the device!
3. It is forbidden to leave the patient unattended. Device should be used only when there is someone who is trained and knowledgeable.
4. Before using the product, it is essential to make sure that there are no damages or different faults which could endanger patient's life. Make sure that all parts are operating correctly and are properly mounted. Moreover, make sure that there are no cracks as well as deformities
5. It is forbidden to use platform for different purposes than it is intended to.
6. It is forbidden to make any changes concerning device's structure due to the fact that it is dangerous and it results in losing warranty.
7. Before using the product, there is necessary to protect the deckchair or bath chair with included straps with Velcro.
8. This device can be only used inside the buildings.
9. The weight of user should not be bigger than maximal weight described on size table and on the data plate.
10. Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.
11. The product contains small parts which children may swallow.
12. Wash with warm water using a soft cloth and mild cleaners. After wet, wipe the base dry. The device should be kept in clean, dry and ventilated room. Maintenance as well as cleaning have to be performed at least once a month.
13. Before using the AKVO base with the AKVOSEGO bath chair or the AKVOLITO bath deckchair, carefully read the user manual of this products.

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The AKVO Base can only be used with the bath chair AKVOSEGO or AKVOLITO. The AKVO Base allows the person using the bathing products to be easily moved between rooms. It also facilitates hygiene activities due to the higher position of the bath chair.

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
For AKVO base which is used with bath chair AKVOSEGO or bath deckchair AKVOLITO : Incorrect positioning of the person with the use of bathing products may lead to deformation of the joints, and long-term keeping of the person in one position significantly increases the risk of pressure ulcers.

GRUPA DOCELOWA DLA WYROBU • INTENDED PURPOSE OF THE PRODUCT WITH A DEFINED TARGET GROUP

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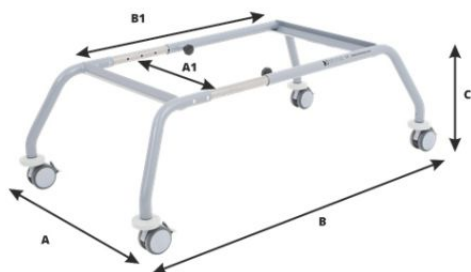For AKVO base which is used with bath chair AKVOSEGO or bath deckchair AKVOLITO : Disabled people (children / adolescents) with cerebral palsy, muscular dystrophy, spina bifida, spinal cord injuries and neurological deficits.

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