

KOD KATALOGOWY	AM-OSK-Z/1R-01
NAZWA HANDLOWA	ORTEZA KOŃCZYNY DOLNEJ
MODEL	ORTEZA KOŃCZYNY DOLNEJ AM-OSK-Z/1R-01
CODE	AM-OSK-Z/1R-01
TRADE NAME	LOWER-EXTREMITY SUPPORT
MODEL	LOWER-EXTREMITY SUPPORT AM-OSK-Z/1R-01



- 1. PL:** Rozepnij ortezę i paski.  
**EN:** Unfasten the brace and straps.



- 2. PL:** Załóż ortezę na nogę i podciągnij do góry w taki sposób, aby rzepka (1) znalazła się w otworze okółorzepkowym, a zegary szyn (2) znajdowały się w osi stawu kolanowego w określonym przez lekarza prowadzącego w zakresie ruchomości stawu. Tylko prawidłowo założona orteza pozostaje w osi stawu biodrowego i przylega równomiernie do całej kończyny. Następnie zapnij rzep (3).

**EN:** Put the brace on your leg and pull it up (1) to keep the kneecap in the patellar hole, and the drop locks of the splints (2) in the axis of the knee joint, only within the range of joint mobility specified by the attending physician. Only a properly fitted orthosis remains in the axis of the hip joint and lies against the entire limb. Fasten the Velcro closures (3).



**3. PL:** Przepleć taśmy przez klamerki i zapnij na rzepy w podanej kolejności (4,5,6,7).

**EN:** Interlace the straps through the buckles and fasten the Velcro straps according to the following steps (4,5,6,7).



**4. PL:** Gotowy wyrób.

**EN:** Product is ready to use.